Table 1

Light intensity (LED-/halogen lamp)	Polymerisation time (per layer)
≥ 500 mW/cm²	20 s A1, A2, A3, A3.5, A4, BL
≥ 800 mW/cm²	10 s A1, A2, A3, A3.5, A4, BL
	40 s OA2
	20 s OA2

INSTRUCTIONS FOR USE

EN

MD EU Medical device

In accordance with DIN EN ISO 4049

Product description

Visalys® Fill is a light-curing, radiopaque nano-hybrid restorative material. Visalys® Fill contains 87% by weight inorganic fillers and is applied with a dentine enamel bond. It is available in screw syringes and caps for direct application.

Shades

A1, A2, A3, A3.5, A4, OA2, BL

Indications

- Class I to V fillings
- Reconstruction of traumatically affected anterior teeth
- Facetting of discoloured anterior teeth
- Correction of shape and shade for better aesthetic appearance
- Locking, splinting of loose teeth
- Repairing veneers
- Filling deciduous teeth
- Core-build-up under crowns
- Composite inlays

Contraindications

Visalys® Fill contains methacrylate and BHT. Visalys® Fill should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group

Visalys® Fill is suitable for use in all patients without any age or gender restrictions.

Performance features

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User

Visalys® Fill should only be used by a professionally trained dental practitioner.

Use

Preparation
Clean the teeth being treated. If applicable, mark occlusal contact points. Bring material to room temperature before use.

Shade selection
Clean the teeth prior to shade selection. The shade is selected in comparison with the tooth while it is still moist.

Cavity preparation

In principle, the cavities should be prepared according to the rules for adhesive restorative treatment; preparation should be minimally invasive in order to protect healthy dental hard tissue. For anterior restorations, angle off the enamel margins. Then clean and dry the cavity. Caries-free lesions in the cervical area do not need to be prepared; thorough cleaning is sufficient in this case.

Ensure that the working field is sufficiently dry. The use of a rubber dam is recommended. It is recommended that shaping aids be used in cases where a portion of the cavity is in the proximal region. It helps to use translucent female parts, wedged in the proximal region. Keeping separation to the minimum makes it easier to shape the proximal contacts and attach a female part. Suitable protection for the pulp should be applied close to the pulp and covered with a layer of stable cement if necessary.

Bonding materials

Visalys® Fill is applied with a dentine enamel bond, using the adhesive technique. Any light-curing bonding materials can be used. Observe the relevant instructions for use during preparation (etching technique) and processing.

Application

Apply the selected shade of Visalys® Fill (see shade selection) in layers of no more than 2 mm thickness; adapt using a suitable instrument and then cure.

Light-curing

Conventional polymerisation devices are suitable for light-curing the material. The polymerisation time per layer (Polymerisation time) as a function of the light intensity (Light intensity) can be found in Table 1.

Bring the light aperture as close as possible to the light-curing unit as otherwise the curing will be less effective. Inadequate curing can cause discoloration and complaints.

Finishing

Restorations can be finished and polished immediately after removing the female parts (e.g. fine and extra-fine diamond burs, abrasive discs). The tooth should then be fluoridized.

Visalys® Fill can be used for direct or indirect inlays using the known procedures. The physical stabilisation values are improved by the usual external (additional) curing.

Warnings, precautionary measures

• Phenolic substances, especially products containing eugenol and thymol, lead to curing failures in restorative composites. Consequently, the use of zinc oxide/eugenol cements or other materials containing eugenol should be avoided in combination with restorative composites.

• Each Visalys® Fill Caps unit should be used for one patient only and should be applied exerting uniform but not excessive pressure.

• Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content)

Strontium aluminium borosilicate glass, barium aluminium borosilicate glass, silicon dioxide, BisGMA, TEGDMA, initiators, stabilizers, pigments

Storage instructions and application method

Storage at 4°C–28°C. Seal the syringes immediately after removing the material in order to avoid exposure to light and subsequent polymerisation. Do not use after the expiry date.

Disposal

Dispose of the product in accordance with local regulations.

Reporting obligation

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of Visalys® Fill must be reported to the manufacturer and the responsible authority.

Note

The Summary of Safety and Clinical Performance of Visalys® Fill can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

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